Moulded Valved Face Mask FFP3 NR

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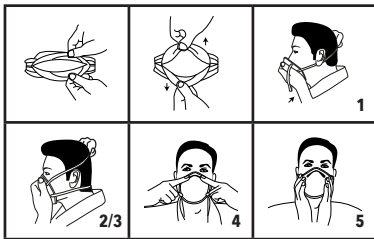
Version date: 17.09.2018

GB Specification

Shelf Life: Expiry date on product or individual packaging
FFP3 NR : Single Use. Do not use for more than one shift
Protection duration: ≤ 8 hrs
Size: Standard

Fitting Instructions

IMPORTANT: To provide the required level of protection, it is essential that the respirator is fitted correctly. Please follow the guidance below:

1. Hold the respirator in your hand with the nosepiece at your fingertips, allowing the elasticated headbands to hang freely below your hand.
2. Press the respirator firmly against your face with the nosepiece positioned on the bridge of your nose.
3. Draw the headbands over your head one at a time. Position the top band high on the back of your head. Position the bottom band just below your ears.
4. Using both hands, mould the metal nosepiece to the shape of your nose.
5. To check the fit, place both hands completely over the respirator and inhale vigorously. If air leaks around the edges of the respirator, readjust the respirator, manipulating the nose bridge and headband to achieve an acceptable seal. If a proper seal cannot be achieved DO NOT enter the contaminated area.

Mark	Meaning
EN149:2001+A1:2009	Respiratory standard number
FFP1 NR	Class 1, low filtering efficiency - Concentrations up to 4 x OEL or 4 x WEL
FFP2 NR	Class 2, medium filtering efficiency - Concentrations up to 12 x OEL or 10 x WEL
FFP3 NR	Class 3, high filtering efficiency - Concentrations up to 50 x OEL or 20 x WEL

Note:

OEL = Occupational Exposure Limit
WEL = Workplace Exposure Limit

Warning

The user should be trained in fitting the respirator and should be aware of the instructions for use prior to putting on the respirator. Use in adequately ventilated areas, which are not deficient of oxygen and do not contain explosive atmospheres. The respirator must be fitted correctly to ensure the level of protection expected is achieved. It is unlikely that the requirements for leakage will be achieved if facial hair passes under the face seal.

Discard and replace the respirator if:

1. The respirator is removed whilst in the contaminated area
2. Clogging of the respirator causes breathing difficulties
3. The respirator becomes damaged

FAILURE TO FOLLOW THE INSTRUCTIONS AND WARNINGS ON THE USE OF THIS RESPIRATOR DURING ALL TIMES OF EXPOSURE CAN REDUCE THE EFFECTIVENESS OF THE RESPIRATOR AND COULD RESULT IN ILLNESS OR DISABILITY.

Storage and Shelf Life

This respirator should be stored in a container, protected from dust, chemicals, moisture, away from direct sunlight and in a clean, cool and dry atmosphere with humidity factor less than 75% and in a temperature between +2°C and +55°C. Failure to observe these requirements may affect shelf life and performance.

Cleaning and Maintenance

Do not carry out any cleaning and maintenance procedures on this product. Do not use this product if it has not been stored correctly and/or is not in a fit state for use. Dispose of the product after use.

Disposal

After use, carefully remove the mask in a clean, still environment and place in a sealable bag prior to disposal.

CE Declaration of Conformity

The undersigned: Mr Darrell Morris

as authorised by: Silverline Tools

Declares that

Identification code: 763615

Description: Moulded Valved Face Mask

Conforms to the following directives and standards:

- PPE Regulation 2016/425
- EN 149:2001+A1:2009

Notified body: Inspec International Ltd, Manchester, England

The technical documentation is kept by: Silverline Tools

Date: 21/08/18

Signed:

Mr Darrell Morris

Managing Director

Name and address of the manufacturer:

Powerbox International Limited, Company No. 06897059. Registered address: Powerbox, Boundary Way, Lufton Trading Estate, Yeovil, Somerset BA22 8HZ, United Kingdom.

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Keuringsinstantie: Inspec International Ltd, Manchester, England
Zastosowana dyrektywa oceny zgodności: Inspec International Ltd, Manchester, England